CANTOR+NISSEL

Rigid Contact Lens Instructions for Use

fig.1

fig.2

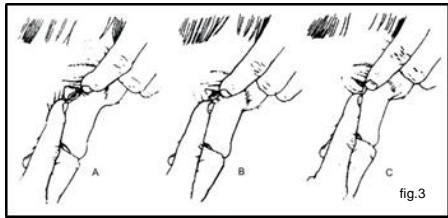
fig.3

Images reproduced by kind permission of Nigel Burnett Hodd, from his book 'Pocket Guide to Contact Lenses' published by Arlington Books, King Street, St James's, London

Lenses should not be worn unless prescribed by a registered optometrist.

CE
0120

[GB]

An important fact to remember with contact lenses is that they are foreign bodies as far as your eyes are concerned. Lens cleanliness and general hygiene are vital.

Always wash your hands before handling your lenses.

These instructions are to be used as a guide in conjunction with any advice given by your optometrist.

Rigid contact lens instructions

Lenses should not be worn unless prescribed by a registered optometrist

The enclosed rigid lens(es) have been supplied to you clean and wet. Please retain them in their sealed mailer or screw top vial until they are required for fitting. If the lens(es) have not been fitted within 28 days, remove them from their packaging; clean and rinse the lens(es) and container and replenish the wetting solution. This procedure should be repeated, dependent upon the life of the solution used.

Removing the lens carefully from its container, place the lens on your index finger, with the inside of the lens looking at you. Steady your hand in front of your face coming from below and use the middle finger to pull the lower lid down and to steady the approach of the index finger. At the same time, use your other hand to pull the top lid up and to inhibit the blink reflex (fig.1). Put the lens gently on the middle of your eye but **DO NOT RELEASE YOUR LIDS**. Now withdraw the finger, release the top lid and then the bottom lid. Blink and the lens should be in place.

Check that the lens has moved onto the centre of the eye by ensuring you can see by covering the eye without the lens with your hand. If the lens does not go in with the first attempt rinse it with wetting solution and repeat the procedure.

Sometimes the lens will displace off the centre of the eye either due to inexperienced insertion or rubbing the eye unwittingly. Re-centring the lens is an easy task even if you do not know where it has gone. If you know where the lens is, look straight into a mirror and turn your whole head away from the side where the lens is lodged whilst still staring into the mirror (fig.2). The lens then becomes more exposed. Now pull the lid back at the point where the lens is lodged and use the lid margin as a pusher to push the lens up onto the cornea. If you do not know where the lens is please seek the advice of your optometrist.

To remove the lens(es) using the two finger method (fig.3), make sure your hands are clean and that your lens case has fresh storage solution in it. Using the forefinger of each hand put one on the middle edge of the top lid and one on the middle edge of the bottom lid. Move the lids apart to free the lens and then press with the fingers to hold the edge of the lid against the eye. Push the lids together so that the lens flips out or just push the top lid down to force the lens to flip out over the bottom lid. If in doubt ask your optometrist to show you how to remove the lens(es).

Once the lens has been removed from the eye, rinse with cleaning solution. Replace the lens into the case containing solution and repeat with the second lens. Your optometrist will advise you on the correct solution to use.

Do's

- Follow carefully the instructions and advice of your Optician concerning cleaning, handling and storage.
- Return for all aftercare visits that your Optician recommends.
- Remove your lens(es) if there is any adverse reaction at all and consult your Optician.
- Handle your lens(es) with care, always washing, rinsing and drying your hands thoroughly first and ensuring any person who may be assisting you does the same.
- Remove your lens(es) before going to sleep, unless otherwise advised by your Optician.

Don't's

- Wear your lens(es) for longer than advised by your Optician.
- Wear your lens(es) when using any eye drops or other eye medication unless advised by your Optician.
- Expose your lens(es) to hairsprays, cosmetics, excessive heat or pressure.
- Wear your lens(es) in the presence of noxious or irritating fumes.

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