## Soft Contact Lens Instructions for Use

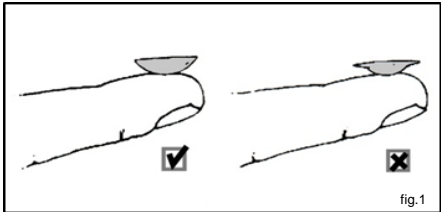
fig.1

fig.2

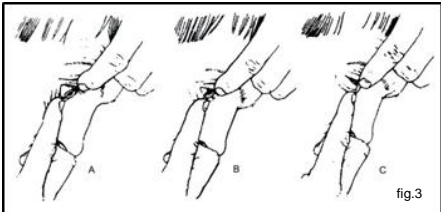
fig.3

Images reproduced by kind permission of Nigel Burnett Hodd, from his book 'Pocket Guide to Contact Lenses' published by Arlington Books, King Street, St James's, London

**Lenses should not be worn unless prescribed by a registered optometrist.**

### [GB]

An important fact to remember with contact lenses is that they are foreign bodies as far as your eyes are concerned. Lens cleanliness and general hygiene are vital.

Always wash your hands before handling your lenses.

These instructions are to be used as a guide in conjunction with any advice given by your optometrist.

Soft contact lens instructions

The enclosed soft contact lens(es) have been supplied to the optometrist sterile in 0.9% Sodium Chloride solution (Saline) prior to fitting. If the lens(es) have not been used by the indicated expiry date, they must be disposed of and not used.

Remove the lens from its case, being careful not to fold it. Rinse the lens with a saline solution and check that it is not inside out. (fig.1)

Looking straight into a mirror, pull down the lower lid and at the same time lower the chin so that the lower half of your eye is more exposed. Gently press the lens over the lower half of the eye before withdrawing your finger (fig.2). If you cannot control your blink reflex the top lid must be held firmly in place by the other hand.

Check that the lens has moved onto the centre of the eye by ensuring you can see by covering the eye without a lens with your hand. If the lens does not go in with the first attempt rinse it with a saline solution and repeat the procedure.

To remove the lens(es) using the two finger method (fig.3), make sure your hands are clean and that your lens case has fresh storage solution in it. Using the forefinger of each hand put one on the middle edge of the top lid and one on the middle edge of the bottom lid. Move the lids apart to free the lens and then press with the fingers to hold the edge of the lid against the eye. Push the lids together so that the lens crumples up and falls out over the bottom lid. This method is best done leaning over a mirror. If in doubt ask your optometrist to show you how to remove the lens(es).

To clean the lens place it in the palm of your left hand and, using the tip of your right index finger, rub it gently with a few drops of cleaning solution (being careful not to catch the lens with your finger nail), after 20 seconds transfer it to the palm of the right hand and rinse it twice with saline solution whilst gently rubbing it. Replace the lens into the case and repeat with the second lens. Your optometrist will advise you on the correct solution to use.

### Do's

- Follow carefully the instructions and advice of your Optician concerning cleaning, handling and storage.
- Return for all aftercare visits that your Optician recommends.
- Remove your lens(es) if there is any adverse reaction at all and consult your Optician.
- Handle your lens(es) with care, always washing, rinsing and drying your hands thoroughly first and ensuring any person who may be assisting you does the same.
- Remove your lens(es) before going to sleep, unless otherwise advised by your Optician.

### Don't's

- Wear your lens(es) for longer than advised by your Optician.
- Wear your lens(es) when using any eye drops or other eye medication unless advised by your Optician.
- Expose your lens(es) to hairsprays, cosmetics, excessive heat or pressure.
- Wear your lens(es) in the presence of noxious or irritating fumes.
- Use tap water to rinse, clean or store your lens(es).

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